




2701 Bartram Road • Bristol, PA 19007  
800-840-9041 • 215-788-2700  
www.gentell.com • info@gentell.com

# Calcium Alginate Rope Dressing

**12" Rope (30.5 cm)**  
6/box  
10 boxes/case  
60/case  
**GEN-13120**

- Rope is easily cut, packed, molded and folded to fit size of wound
- Uniformly absorbs, collects and contains up to 20 times its weight in wound exudate
- Fluid wicks vertically into dressing minimizing the chance for maceration

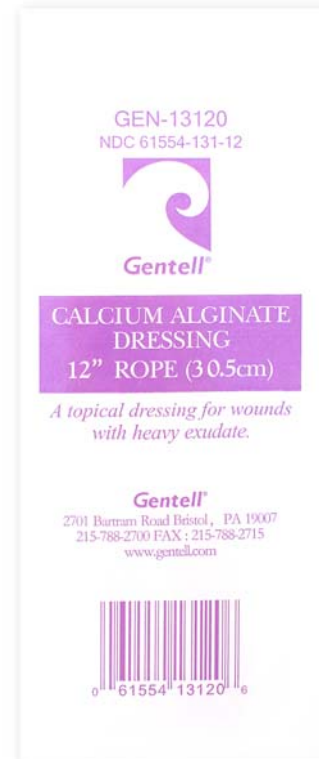

**Gentell's Calcium Alginate Rope Dressing** is a sterile, comfortable, advanced fiber-structured alginate with a highly absorbent capacity. Alginate dressings absorb, collect and contain exudate while providing a moist healing environment. A reaction between the calcium in the dressing and the sodium in the wound exudate creates a gel-like substance that promotes moist wound healing.

## Directions

1. Gentell's Calcium Alginate Rope should be applied in dry form, and should be used on deep cavity wounds such as leg ulcers, pressure ulcers, diabetic foot ulcers and surgical wounds with heavy drainage.
2. Flush the wound with Gentell Wound Cleanser and gently dry the skin surrounding the wound site.
3. Place the Gentell Calcium Alginate Rope in the wound bed. Remember do not cut or pull apart Alginate so the dressing does not become fibrous and frayed. It is much more difficult to remove many small pieces opposed to one large piece.
4. Apply appropriate size cover dressing using a Gentell Bordered Gauze, Comfortell, or Gentell LoProfile Foam.
5. Repeat daily or as ordered.